



INFORMATION IS POWER

SOUNDWAVE™ 2.2

Datalogging Release

Accessing significant data on your patients' program and battery power usage



Advanced Bionics



SOUNDWAVE™ 2.2

Datalogging Release



The all-new datalogging feature in AB's SoundWave™ fitting software provides invaluable insights to help you understand how your patients use programs, features, and accessories.

SoundWave is also the only fitting software that logs data on battery usage and provides battery life estimation.

This important information may help you counsel your patients on the use of the system and evaluate any programming changes that might improve their hearing outcomes.

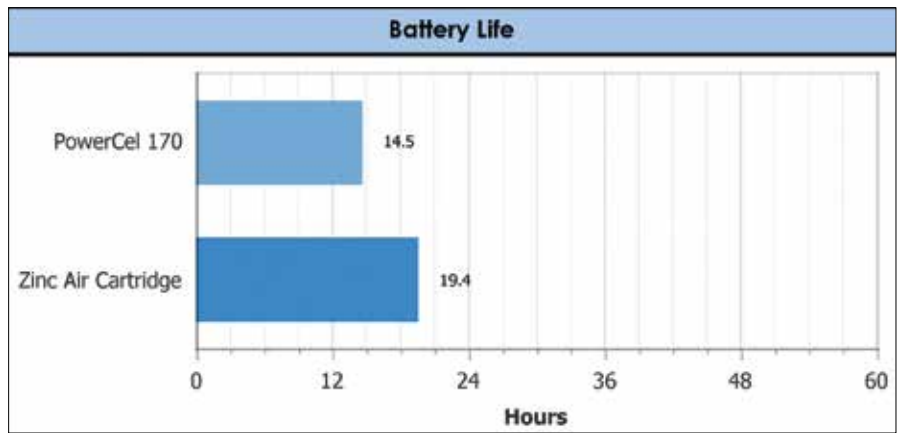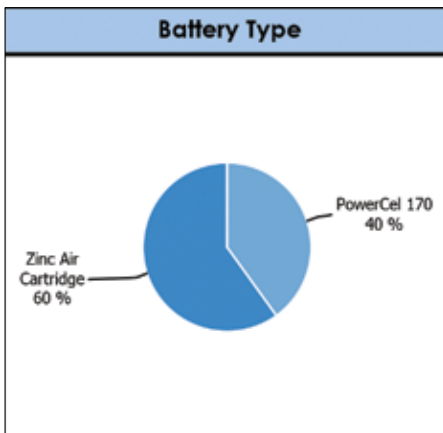


Datalogging helps you determine how your patients are using their Naída CI Q70 processor on a daily basis, including which programs, features, and volume settings they select.

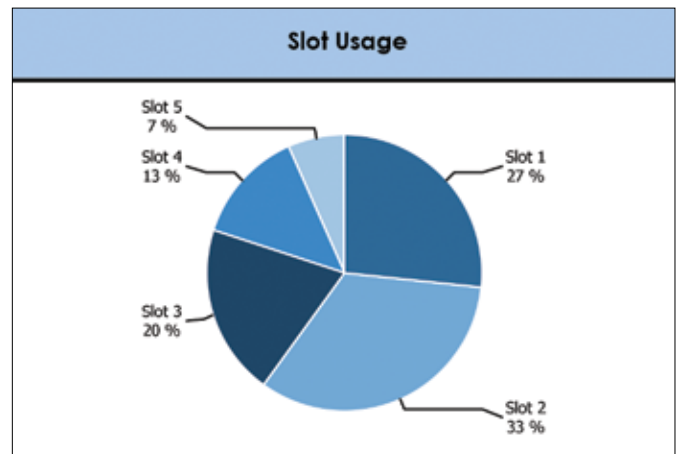
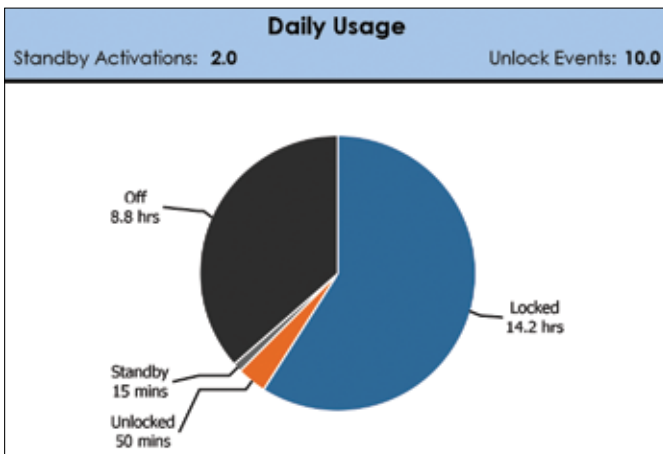


SOUNDWAVE™ 2.2

Datalogging Release

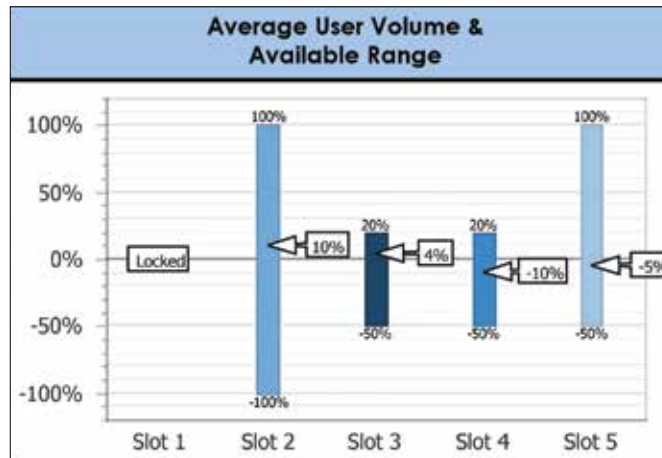

Two graphs indicate the type and loudness of the environments that your patients experience throughout the day. Insight into which acoustic settings your patients encounter can assist with recommending sound processing programs, features, and accessories to help improve speech understanding and music enjoyment.

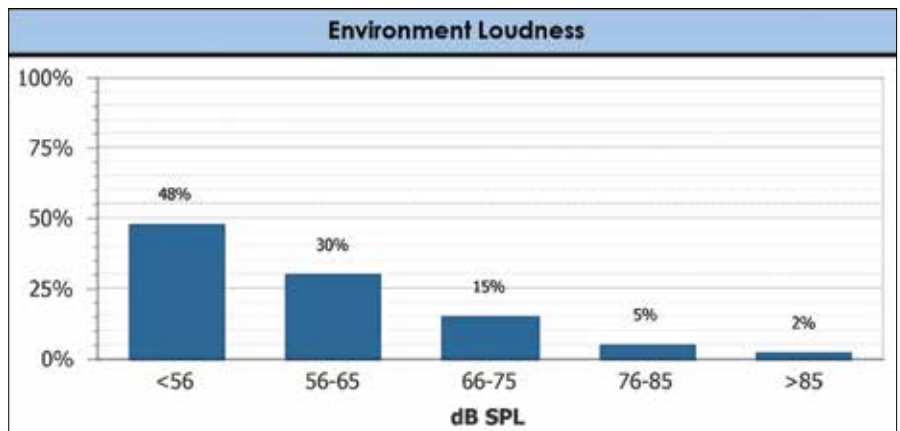
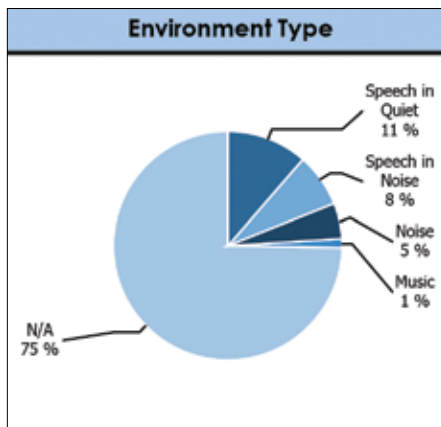


You will also be able to determine the Phonak feature usage, including the ComPilot accessory, as well as UltraZoom, DuoPhone, and ZoomControl processing designed for better speech discrimination in noise and on the phone.

SOUNDWAVE™ 2.2

Datalogging Release

The ComPilot usage graph offers a detailed look at how your patients have been using other devices, including FM and phones. The feature also records whether your patients use the processor or remote control to change volume or programs.

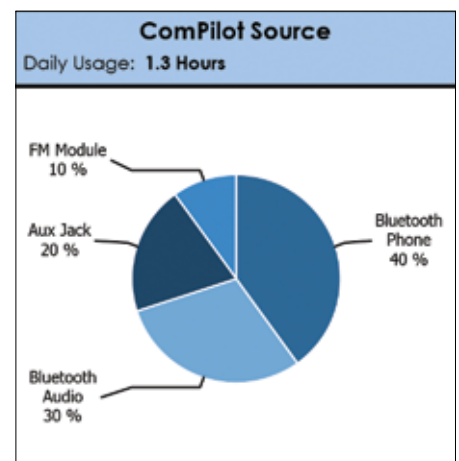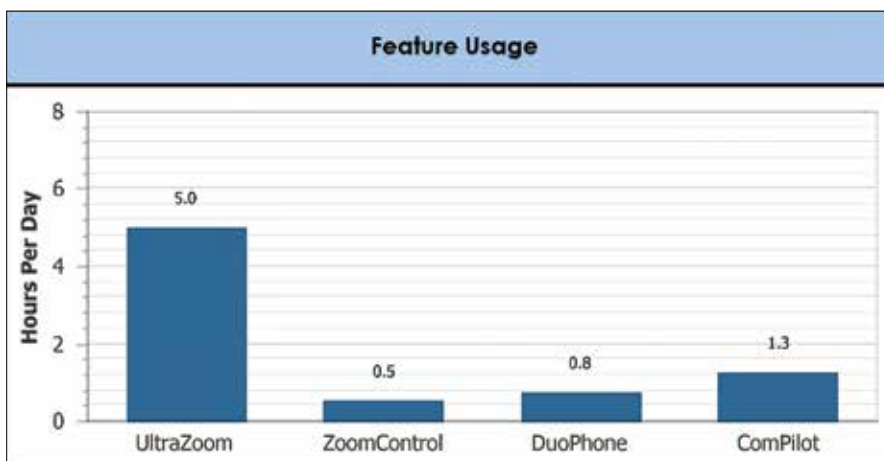

AB's innovations are always designed to help patients hear the most of their world. By providing the very best tools available to the healthcare professionals who support them, we strive to make sure they experience the *best possible journey to hearing with AB*.

To learn more about datalogging with SoundWave from AB,
please contact your local AB representative or visit AdvancedBionics.com/contact.