Advanced Bionics AG
Laubisrütistrasse 28
8712 Stäfa, Switzerland
+41 58 928 78 00

Manufactured by:
Advanced Bionics LLC
California, U.S.A.
+1 661 362 1400

AdvancedBionics.com

029-M529-85 Rev A

© 2015 Advanced Bionics AG. All Rights Reserved.

Naída CI

By Advanced Bionics

AQUACASE™ Container
CI-7431

instructions for use

English, Français, Español, Deutsch, Nederlands, Português

Labeling symbols and their meanings:

	European Community Mark of Conformity. Authorized to affix the CE Mark in 2014	
	Model number	 Date of manufacture
	Lot Number	 Manufacturer
	Type of Protection: B	 Warning
	See Instructions for Use	
	Store at temperatures between -20°C (-4°F) and +55°C (131°F)	
	Suitable for atmospheric range between 70kPa and 106kPa, which is equivalent to 3000m above sea level to 380m below sea level.	
	Suitable for exposure to relative humidity between 0-95%	
IP68	This rating indicates that the Naída CI AquaCase is protected against the following conditions: <ul style="list-style-type: none"> • Failure due to dust penetration • Failure due to continuous immersion in water up to three meters (9 feet, 10 inches) 	

Warnings

- The AquaCase O-Ring should be inspected prior to each use and will need to be changed periodically. If damage to the O-Ring is observed or suspected, replace prior to resuming use. **Do not continue use of the AquaCase if it is damaged.** Doing so may void the warranty and may compromise system performance. If used improperly or damaged, you may be liable for the repair cost of the product. This includes but is not limited to water or particulate matter damage.
- When the AquaCase is moved from a hot environment (i.e. a sauna or whirlpool) to a colder environment (i.e. a pool), condensation may form inside the case. If condensation occurs, remove the processor and place in the drying system provided in your processor kit overnight to ensure proper drying of the device.
- The AquaCase should be kept closed when not in use to prevent the O-Ring from drying out.
- **Following exposure to particulate matter, such as sand or dirt, clean the outside of the case with water and a dry cloth prior to opening.**
- **The AquaCase and O-Ring should be rinsed following submersion in salt or chlorine treated water.**
- Do NOT store the AquaCase in direct sunlight or inside a vehicle during extremely hot or cold weather. The O-Ring should also not be left in direct sunlight. To prevent excessive sun exposure, cover the AquaCase and O-Ring with a towel.

- **When sunbathing or after water use, please remove the Naída CI from the AquaCase or cover the Naída CI/AquaCase to shade it from direct sunlight and prevent excessive temperatures.**
- Do NOT open or close the AquaCase while in a water environment.
- If water or particle ingress is observed, cease use of the AquaCase immediately, remove the processor from the enclosure and place in the drying system provided in your processor kit overnight to ensure proper drying of the device.
- Cables are not intended to be removed or replaced while the AquaCase is submerged. If the cable accidentally detaches during water use, remove the AquaCase from the water and shake excess water from the case prior to reconnecting the cable.
- To prevent damage to Naída CI accessories (e.g. ComPilot, AB myPilot, Listening Check), it is important to cease use prior to engaging in activities in or near water. Once in a dry environment, the recipient may resume use of the AB myPilot with the processor in the AquaCase.
- Due to the possibility of neck injury, use of the lanyard is not recommended during rough play or around moving machinery.
- To avoid an increased risk of strangulation, do not use cables longer than 81 cm (32 inches) for young children.

Notes

- Ensure that the processor is powered on and the selected program supports headpiece (AquaMic) microphone input. If the program does not support headpiece microphone input, select the appropriate slot setting that does. If you are unsure, contact your hearing healthcare specialist for programming assistance.
- Please note that you may notice a slight sound quality difference when switching between your BTE microphone program and your headpiece microphone program.

Naída CI AquaCase™

The AquaCase™ container has been designed for use with the Naída CI during swimming, bathing and exposure to other wet or rugged environments.

The AquaCase is intended for use by unilateral or bilateral Advanced Bionics Naída sound processor recipients who have the ability to notify a parent/caregiver or have the ability to remove the device on their own, if there is a change in sound quality.

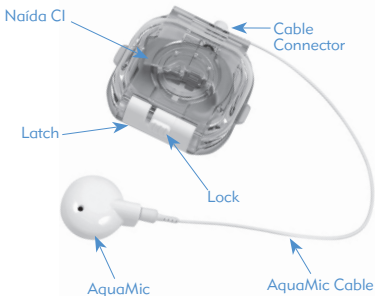
These instructions for use are designed to help recipients, parents, and caregivers understand the use and care of the AquaCase. To ensure proper installation of the AquaCase, recipients with limited strength or ease of motion may require assistance from a guardian or caregiver to achieve a tight seal and proper handling of the AquaCase in accordance with the instructions.

An ingress protection, or IP, rating is used to determine

the strength of an enclosure surrounding the electronic equipment. Each IP rating consists of two numbers. The first number means the level of protection an enclosure provides against the entrance of solid objects, materials, or dust. The second number means the level of protection against the entrance of water.

The AquaCase, AquaMic, and AquaMic™ headpiece cable carry a rating of IP68. This rating means that that the AquaCase system when using these parts is protected against the following conditions:

- Complete protection against the entrance of dust
- Failure due to continuous use in water up to three meters (9 feet, 10 inches)


Inserting the Naída CI in the AquaCase

The AquaCase has a lock to make sure the case is not accidentally opened, allowing moisture in and damaging the Naída CI. The lock is located on the front of the AquaCase.

 ***Do NOT open or close the AquaCase while in a water environment.***

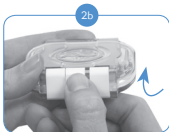While holding the lock in the detached position, pull the latch open with your fingertips.


Before placing the Naída CI in the AquaCase, visually inspect the O-Ring seal to make sure it is placed correctly. **Visually confirm that the O-Ring is not dry, brittle or dirty. If it is, replace O-Ring prior to use.**


 ***If damage to the O-Ring is observed or suspected, replace prior to resuming use.***

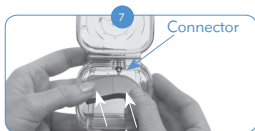- 4** The cable and headpiece must be removed from the Naída CI before inserting into the case; however, the T-Mic2 /earhook may remain attached to the processor.

- 5 Only the PowerCel 110 battery and AquaMic may be used with the AquaCase.
- 6 Make sure the processor is powered on and the selected program supports headpiece (AquaMic) microphone input. Contact your hearing healthcare specialist for programming assistance.

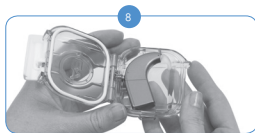
Line up the processor headpiece cable connector (located on the back spine of the Naída CI below the Program Button) with the cable connector located inside of the AquaCase. Slide the Naída CI into the AquaCase until the connection is made.

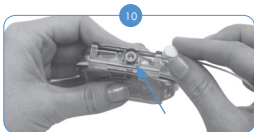The Naída CI should fit tightly to the back wall of the AquaCase and lie flat against the bottom surface.


- 9 Once the Naída CI is in place, close the lid of the AquaCase. Return the latch to the closed position. The lock will automatically close, making sure the AquaCase is locked.

Connect the AquaMic headpiece to the AquaMic cable. For more information, refer to the AquaMic 'Instructions for Use'. Push the AquaMic cable connector firmly into the AquaCase connector.

 ***To avoid an increased risk of strangulation, do not use cables longer than 81 cm (32 inches) for young children.***

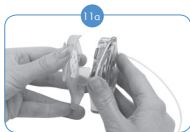 ***Cables are not intended to be removed or replaced while the AquaCase is submerged. If the cable accidentally detaches during water use, remove the AquaCase from the water and shake excess water from the case prior to reconnecting the cable.***

The AquaCase can be worn in different locations using the Clip or Lanyard. Cables come in different lengths to allow you to customize your wearing preference.


 ***Due to the possibility of neck injury, use of the lanyard is not recommended during rough play or around moving machinery.***

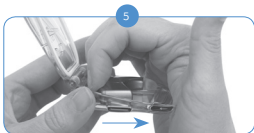Removing the Naída CI from the AquaCase

- 1** Make sure that the AquaCase is towel-dried before removing the processor to prevent water from accidentally getting on the processor.

 ***Following exposure to particulate matter, such as sand or dirt, clean the outside of the case with water and a dry cloth prior to opening.***

- 2 To open, unlock the AquaCase by sliding and holding the lock.
- 3 While holding the lock use your fingertips to pull open the latch and open the AquaCase.
- 4 Be careful not to allow water from your hair or body to fall in the AquaCase or on the Naída CI.

To remove the Naída CI, carefully slide the processor forward, making sure not to pull up until Naída CI is free from the connector.

Remember to utilize the drying system after any water related activities.

- 6 Continue use of the Naída RF cable and UHP.

Replacing the O-Ring on the AquaCase

The AquaCase O-Ring should be inspected prior to each use and will need to be changed periodically. If damage to the O-Ring is observed or suspected, replace prior to resuming use.

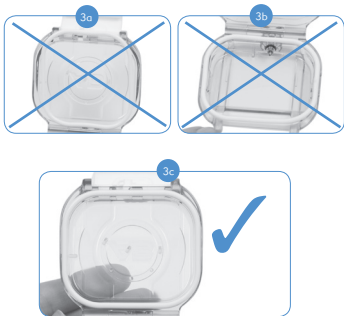
To remove the existing O-Ring, use your fingers to lift a portion of the old O-Ring out of the AquaCase.

Once a portion of the O-Ring is lifted out, hold the O-Ring and pull the remaining portion out of the AquaCase mounting groove.

The AquaCase and O-Ring should be rinsed following submersion in salt or chlorine treated water.

To insert the new O-Ring, place a portion of the new O-Ring in the AquaCase mounting groove on the case. Take care to match the rectangular shape of the O-Ring correctly with the rectangular shaped AquaCase.

While holding the O-Ring in the groove with one hand, carefully stretch the remaining portion of the O-Ring into the remaining groove.

Run your thumb over the O-Ring to make sure of proper insertion.

Extreme care should be taken to make sure the O-Ring is level and secure in the mounting groove.

Care and Maintenance

Clean with a damp cloth. Do not use solvents. Remove the O-Ring prior to cleaning.

Operating Conditions	
Operating Temperature	0°C - 45°C
Storage Temperature	-20°C - 55°C
Relative Humidity	Up to 95%
Altitude	3000m above sea level and 380m below sea level

Naída CI External Equipment and Accessories

The products listed below are compatible

Description	Model Number
Naída CI Q90 Sound Processor*†	CI-5280-120, CI-5280-130, CI-5280-140, CI-5280-150, CI-5280-160, CI-5280-170, CI-5280-190, CI-5280-110, CI-5280-220, CI-5280-230, CI-5280-240
Naída CI Q70 Sound Processor*†	CI-5245-110, CI-5245-120, CI-5245-130, CI-5245-150, CI-5245-160, CI-5245-170, CI-5245-140, CI-5245-190, CI-5245-220
Naída CI Q30 Sound Processor*†	CI-5260-120, CI-5260-140, CI-5260-150
AquaMic*	CI-5306
PowerCel 110*	CI-5511-110, CI-5511-120, CI-5511-130, CI-5511-140, CI-5511-150, CI-5511-160, CI-5511-170, CI-5511-190, CI-5511-220
PowerCel 110 Mini*†	CI-5521-240

AquaMic Cable*	CI-5414-201, CI-5414-202, CI-5414-203, CI-5414-204, CI-5414-205, CI-5414-206, CI-5414-301, CI-5414-302, CI-5414-303, CI-5414-304, CI-5414-305, CI-5414-306, CI-5414-401, CI-5414-402, CI-5414-403, CI-5414-404, CI-5414-405, CI-5414-406, CI-5414-501, CI-5414-502, CI-5414-503, CI-5414-504, CI-5414-505, CI-5414-506
AquaCase Lanyard, Standard	CI-7430-010, CI-7430-020
AquaCase O-Rings	CI-7320
AquaCase Clip	CI-7319

*Applied part per IEC 60601-1

¹Only available in markets where regulatory approval has been received and the product has been made commercially available. Contact Advanced Bionics for more information.

NOTE: model number extensions denote variants related to product color, size, and/or length.

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