




Marvel CI provides numerous ways for recipients to stream audio from their favorite devices and connect with the ones they love. With Marvel CI, recipients can stream using Bluetooth®, Roger™, and AirStream technologies. This clinical resource is designed to identify which streaming options are available for the recipient based on which hearing instrument the recipient is using with Marvel CI.

NAÍDA CI M90 on one side

	BLUETOOTH	ROGER	AIRSTREAM	
RECOMMENDED ON THE OTHER SIDE	Naída CI M90	Bilateral/Bimodal streaming via integrated Bluetooth	Bilateral/Bimodal streaming with Phonak TV Connector and Phonak PartnerMic	
	Phonak Naída Link M	Bilateral/Bimodal streaming (Roger mic input only)* Microphones: Roger iN or Roger* Receivers: Roger installed into each device via Roger iN microphone or via Roger X + Roger installer		
OTHER HEARING INSTRUMENTS	Sky CI M90	Unilateral streaming to the selected hearing instrument via integrated Bluetooth	Bilateral/Bimodal streaming with Phonak TV Connector.	
	Phonak Paradise or Phonak Marvel HI		Unilateral streaming to the paired hearing instrument with Phonak PartnerMic	
	Naída CI Q90	Unilateral streaming to Naída CI M90 via integrated Bluetooth OR To Naída CI Q90 via Naída CI Connect, Roger 17, or Phonak ComPilot	Bilateral streaming (Roger mic input only)* Microphones: Roger iN or Roger* Receivers: Roger installed into Naída CI M90 via Roger iN microphone or via Roger X + Roger installer and Roger 17 for Naída CI Q90	Bilateral streaming with Phonak TV Connector when Naída CI Connect is attached to Naída CI Q90. Unilateral streaming to Naída CI M90 with Phonak PartnerMic
	Phonak Naída Link Q	Unilateral streaming to Naída CI M90 via integrated Bluetooth OR To Phonak Naída Link Q via Roger 10, Roger 15 or Phonak ComPilot	Bilateral streaming (Roger mic input only)* Microphones: Roger iN or Roger* Receivers: Roger installed into Naída CI M90 via Roger iN microphone or via Roger X + Roger installer and Roger 10 or Roger 15 attached to Naída Link Q	Unilateral streaming to Naída CI M90 via Phonak TV Connector and Phonak PartnerMic
	Chorus	Unilateral streaming to Naída CI M90 via integrated Bluetooth OR To Chorus via integrated Roger	Bilateral streaming (Roger mic input only)* Microphones: Roger iN or Roger* Receivers: Roger installed into Naída CI M90 via Roger iN microphone or via Roger X + Roger Installer. Roger 02 is integrated into Chorus.	Unilateral streaming to Naída CI M90 via Phonak TV Connector and Phonak PartnerMic

*Bluetooth on Roger microphone should be disabled because Phonak Marvel and Paradise technologies have integrated Bluetooth.

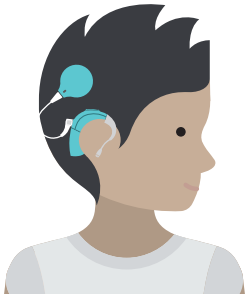

SKY CI M90 on one side

	BLUETOOTH	ROGER	AIRSTREAM	
RECOMMENDED ON THE OTHER SIDE	Sky CI M90	Bilateral/Bimodal streaming via integrated Bluetooth	Bilateral/Bimodal streaming with Phonak TV Connector and Phonak PartnerMic	
	Phonak Sky Link M	Bilateral/Bimodal streaming (Roger mic input only)* Microphones: Roger iN or Roger* Receivers: Roger installed into each device via Roger iN microphone or via Roger X + Roger installer		
OTHER HEARING INSTRUMENTS	Naída CI M90	Unilateral streaming to the selected hearing instrument via integrated Bluetooth	Bilateral/Bimodal streaming with Phonak TV Connector.	
	Phonak Paradise or Phonak Marvel HI	Bilateral/Bimodal streaming (Roger mic input only)* Microphones: Roger iN or Roger* Receivers: Roger installed into each device via Roger iN microphone or via Roger X + Roger installer	Unilateral streaming to the paired hearing instrument with Phonak PartnerMic	
	Naída CI Q90	Unilateral streaming to Sky CI M90 via integrated Bluetooth OR To Naída CI Q90 via Naída CI Connect, Roger 17, or Phonak ComPilot	Bilateral streaming (Roger mic input only)* Microphones: Roger iN or Roger* Receivers: Roger installed into Sky CI M90 via Roger iN microphone or via Roger X + Roger installer and Roger 17 for Naída CI Q90	Bilateral streaming with Phonak TV Connector when Naída CI Connect is attached to Naída CI Q90. Unilateral streaming to Sky CI M90 with Phonak PartnerMic
	Phonak Naída Link Q	Unilateral streaming to Sky CI M90 via integrated Bluetooth OR To Phonak Naída Link Q via Roger 10, Roger 15 or Phonak ComPilot	Bilateral streaming (Roger mic input only)* Microphones: Roger iN or Roger* Receivers: Roger installed into Sky CI M90 via Roger iN microphone or via Roger X + Roger installer and Roger 10 or Roger 15 attached to Naída Link Q	Unilateral streaming to Sky CI M90 via Phonak TV Connector and Phonak PartnerMic
	Chorus	Unilateral streaming to Sky CI M90 via integrated Bluetooth OR To Chorus via integrated Roger	Bilateral streaming (Roger mic input only)* Microphones: Roger iN or Roger* Receivers: Roger installed into Sky CI M90. Roger 02 is integrated into Chorus	Unilateral streaming to Sky CI M90 via Phonak TV Connector and Phonak PartnerMic