

Designed for **EASIER**, **FASTER**, **SMARTER** fitting, Target CI offers an efficient and intuitive experience. Use the *keyboard shortcuts* below to save time and clinical effort when fitting.

KEY	ACTION
GENERAL	
F1	Web help *will be enabled in Target CI 1.1
Alt + F4	Close Window (if there is a "x" to close the window)
Esc	Close Window (if there is a "Cancel" button)
Enter	Close window with default action (e.g. "Ok" button) or navigate to next wizard step
Ctrl + Tab	Navigate to next screen on current major section (Client / Instruments / Fitting) *selection of tab type will navigate through those screens
Shift + Ctrl + Tab	Navigate to previous screen on current major section (Client / Instruments / Fitting) *selection of tab type will navigate through those screens
Ctrl + P	Open reports
Ctrl + S	Save session
Ctrl + W	Save & close session
MAPPING NRI	
Spacebar	Stop All Stimulation
Page Down	Adjust All Markers Down
Page Up	Adjust All Markers Up
Down Arrow	Adjust Markers Down (M, T, Clipping)
Up Arrow	Adjust Markers Up (M, T, Clipping)
L or Ctrl + Left Arrow	Bilateral Window: Focus Left Side
R or Ctrl + Right Arrow	Bilateral Window: Focus Right Side
0-9	Data Entry
Alt + Left Arrow	Move Entire Channel Selection to the Left
Alt + Right Arrow	Move Entire Channel Selection to the Right
Left Arrow	Move Selection Left
Right Arrow	Move Selection Right
C	Select Clipping Marker
G (goes to Map Input Gain tab)	Select Gain Marker
H	Select High Markers (NRI)
L	Select Low Markers (NRI)
M	Select M Marker
T	Select T Marker
Shift + Left Arrow	Select Multiple Channels to the Left
Shift + Right Arrow	Select Multiple Channels to the Right
S	Toggle Stimulation On/Off
Ctrl + Z	Undo
Ctrl + Y	Redo
Shift + Tab	Move Numerical Data Entry Selection Left
Tab	Move Numerical Data Entry Selection Left
Ctrl + A	Select All Markers
FINE TUNING	
Ctrl + Z	Undo
Ctrl + Y	Redo
AUDIOGRAM DIRECT	
Space	Play Signal
Enter	Save point
T	Talk to client
E	Enable HL > 100dB