

Make the Fun Last!

CARE TIPS FOR YOUR WATERPROOF ACCESSORIES

So many of our favourite activities take place in active settings—swimming, gardening, sports, and exercise class. Advanced Bionics waterproof solutions give you the same great hearing while you enjoy even the most rugged activities, all while providing secure, worry-free wearing. Follow these simple guidelines for taking care of your equipment when around water:

Check the O-Ring

Make sure the O-Ring is clean and seated in your accessory prior to using. Check the O-Ring regularly and replace it if it starts to turn yellow or hard.

Rinse off waterproof accessories

After water use, make sure to rinse off the Slim HP AquaMic™ headpiece and other waterproof accessories under clean water and shake off any excess.

Dry all the components

After water use, shake off any excess water from all water accessories, checking every port including the USB charging port. Wipe off any remaining water with a clean and dry cloth. Remove your sound processor from the M Waterproof Battery and put your SlimHP AquaMic in the Drying Kit, or use a compressed air can to ensure all humidity is absorbed. Ensure the accessories are fully dried before charging.

