# QUICK START GUIDE

HARMONY™ *Sound Processor*

English/Français/Español


Quick Start Guide for the Auria™ Harmony™ Processor

# English

Read and understand the entire user guide for the *Auria Harmony Processor* before using the processor. Printed user guides are available upon request.

### Auria™ Harmony™ Processor Parts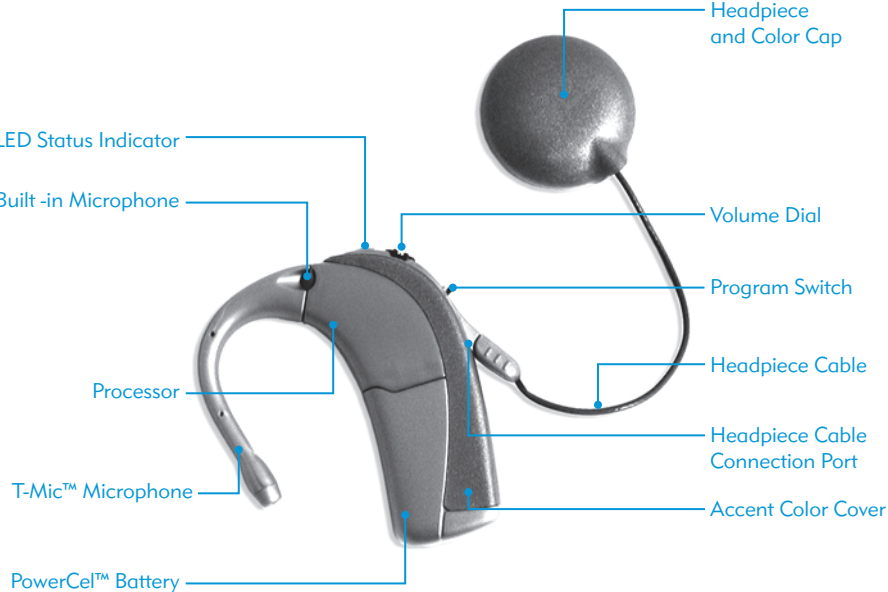


### Charging Your PowerCel Batteries

**Step 1:**  
Place the charger on a flat surface.


**Step 2:**  
Connect the appropriate power adapter to the power supply.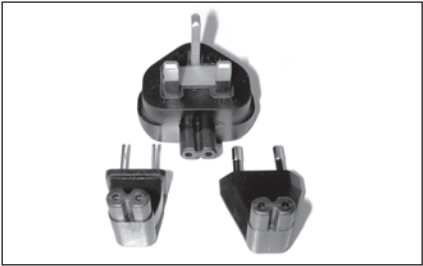

### Step 3:

Connect the cable from the power supply to the charger.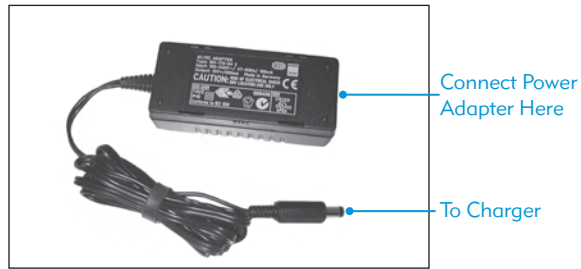


### Step 4:

Insert the power adapter that you connected to the power supply into an outlet. A green light on the charger will light up when there is a proper connection.

### Step 5:


Slide the PowerCel onto the charger. You can charge up to four PowerCels at once. There are two indicator lights for each PowerCel slot. The orange light indicates that the PowerCel is charging. This light will turn off when the PowerCel is fully charged. The PowerCel recharges in 4–5 hours. A red light indicates that there may be a problem with the PowerCel.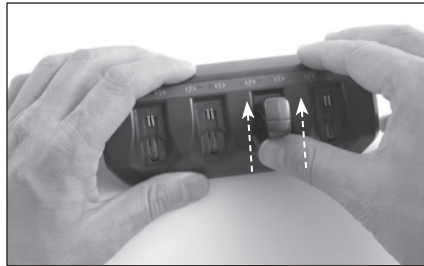

## Assembling Your Harmony Processor

### Step 1:

To attach the earhook, simply align and push into place.

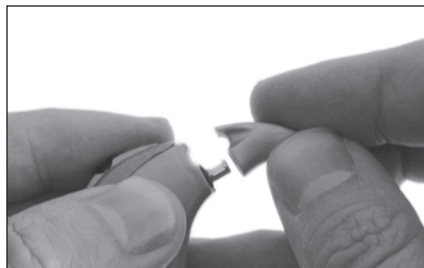### Step 2:

Slide a fully charged PowerCel onto the processor. This will turn on the processor. The ORANGE LED Status Indicator should flash two or more times to verify that the PowerCel is sufficiently charged.

### Step 3:

To attach the chosen Color Cover (optional), simply align and press into place.

### Step 4:


Attach the headpiece cable by sliding it into the headpiece cable connection port of the processor.

## Wearing Your Harmony Processor

### Step 1:

Set the program switch to the appropriate program and turn the volume down by moving the volume dial in a counterclockwise direction.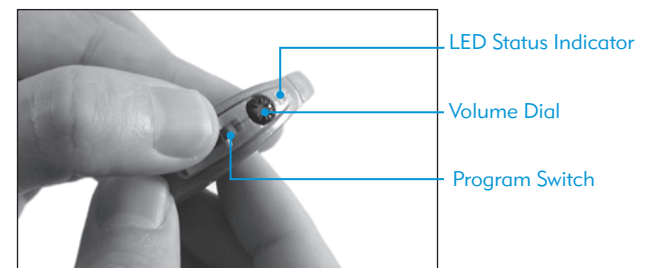

### Step 2:

Place the processor comfortably in position behind the ear.

### Step 3:

Place the headpiece over your implant (you will feel the magnetic attraction).

### Step 4:

Adjust volume to a comfortable level.

### Step 5:

If set by the audiologist (typically for children or those with caregivers), the LED Status Indicator will verify the lock status (the red light will flash rhythmically with loss of lock) and microphone status (the green light should flicker when snapping your fingers or when speaking in a loud voice near the microphone).

### Step 6:

If necessary, perform a behavioral listening check, for example say a few words or check if the Ling 6 sounds (ah, oo, ee, s, sh, mm) can be detected/repeated.

## Operating Your Harmony Processor

### Program Switch

The Harmony can store up to three programs (P1, P2, P3). The switch has three positions to represent each program. P1 is at the bottom, P2 is in the middle, and P3 is at the top.

### Volume Control

The volume control adjusts the loudness level of the program. The volume position is indicated by the white dot located on the dial.

### Built-in Telecoil

If enabled by the audiologist, the Built-in Telecoil offers access to private and public loop systems in some cinemas and meeting rooms and with hearing-aid-compatible telephones.

## Connectivity Options

### Standard Earhook

When the Standard Earhook is used, the built-in microphone located at the top of the processor is the primary source for capturing sound.

### T-Mic™ Microphone

Many users find that the T-Mic provides superior everyday performance especially in localizing sounds and in noisy environments. It can be used easily with headphones and telephones, including mobile phones.

### Auria Direct Connect Earhook

This earhook is a quick way to connect to the external T-coil and with the Direct Connect Cable and Audio Interface Cable, to battery-powered audio devices such as MP3 players, CD players, FM systems, and other assistive listening devices.

### Auria iConnect™ Adapter

The iConnect provides cable-free connection to miniature FM receivers worn directly at ear level.

## Light Emitting Diode (LED) Status Indicator

### PowerCel Charge Status

When a PowerCel is inserted onto the processor, the LED will blink as follows:

- 4 quick ORANGE blinks indicate that the PowerCel is fully charged
- 2–3 quick ORANGE blinks indicate that the PowerCel is sufficiently charged to power the system
- 1 quick ORANGE blink indicates that the PowerCel charge is nearly depleted
- If the Orange LED doesn't light up at all, the PowerCel is completely depleted

### Lock Status (programmable feature)

The term lock refers to the successful communication between your processor and implant. The PowerCel charge status sequence will be followed by a RED light that will begin to flash approximately once per second until lock is achieved. Once your headpiece is properly positioned on your head, and information is transmitting to your implant, the flashing will stop. In addition, if at any time during regular use the Harmony processor loses lock with your implant the RED LED will flash approximately once per second until lock is re-established.

### Microphone/System Status (programmable feature)

When the battery and lock sequences are complete, the light should flicker GREEN with loud inputs (e.g., when you snap your fingers or speak near to the headpiece microphone) to indicate normal operation.

### Low Battery and Change Battery Indicators (programmable feature)

When the PowerCel is near depletion but still able to support stimulation, a steady ORANGE light (low battery indicator) will illuminate indicating that it is time to change the PowerCel. When the PowerCel is depleted to the point that it is unable to support stimulation but not completely depleted, the ORANGE light will blink twice every 3 seconds (change battery indicator). Once the PowerCel is fully depleted, no light will emit from the processor, and no sound will be transmitted to your implant.

**NOTE:** The Microphone/System Status, Low Battery Indicator, Lock Status, and Change Battery Indicator are features that are programmed by your audiologist. Please consult with your audiologist to determine if these features have been enabled for your Harmony processor in any or all of your programs.

### Processor Error Indicator

In rare instances, your Harmony processor LED may emit a continuous RED light. This indicates a potential problem with your Harmony processor. If this should occur, first remove and replace your power source. If this does not resolve the problem, please contact your audiologist for additional troubleshooting.

## Care and Maintenance

- The Auria Carrying Case is provided to keep the Harmony processor, accessories, and batteries safe while not in use.
- The Harmony processor and headpiece can be cleaned with a lightly dampened cloth or tissue.
- The battery contacts on the rechargeable battery and the processor should be kept free from dirt and dust.
- Clean the contacts with a soft bristle brush at least once a month. Immediately after exposure to moisture or perspiration, clean the contacts with a dry lint-free cloth.
- A dri-aid system is provided with the carrying case. Its regular use prevents moisture build-up in the electronic components.
- All parts of the Harmony processor have been designed to tolerate rain, moisture, sweat and dust. However, we advise that the processor and headpiece be removed during bathing, showering, or swimming.

Should you have any problems or questions about your Harmony processor, please refer to your *User Guide for the C1 Auria Harmony Processor*. Additional assistance and support is available by contacting your center or AB.

### WARNINGS:

The PowerCel used in this device may present risk of fire or chemical burn if mistreated. Do not disassemble, heat above 100°C / 212°F or incinerate. Dispose of used PowerCels appropriately. Never put PowerCels in your mouth. If batteries are swallowed, contact your physician or local poison control center immediately.

To prevent injury or burns due to a short circuit, do not allow metal objects, such as keys or coins, to contact the battery terminals. A wallet is provided to carry and protect your PowerCels.

While no injury cases have resulted, components of the system may become hot during normal use or a fault condition. If a particular device's temperature results in discomfort or pain, promptly remove the product and contact your local Advanced Bionics representatives.

The Direct Connect Cable should only be used with battery-operated devices. Devices powered by direct AC current (i.e., devices that are plugged directly into a wall outlet) require an interface cable with appropriate safety protection.

Power supplies and battery chargers should be operated in an open area to ensure adequate airflow. While no injury cases have resulted, components may become hot during normal use or a fault condition. If the device's temperature results in discomfort or pain when touched, disconnect the power source and contact your local Advanced Bionics representatives.

Small parts hazard: Parents and caregivers should be counseled that the processor, Auria T-Coil, Accent Color Covers, Accent Color Headpiece Caps, the metal contact on the inside of the color cap, earhooks, batteries, and the battery compartment of the iConnect may cause choking if swallowed.

Your processor or headpiece should be serviced only at AB. Do not attempt to open or repair the processor. Do not continue to use the processor if any part of it is damaged. Unauthorized opening of the processor, or other equipment, will void the warranty and may compromise system performance.

Your processor has been programmed especially for you. Do not attempt to use any sound processor that has not been programmed specifically for you. Doing so may provide suboptimal sound information and could potentially cause physical discomfort.

## Table of Common LED Status Indicator Indications


(A complete description of all indications can be found in the *User Guide for the Auria Harmony Processor*)

Color	Behavior	Indication
Orange	Blinks when a PowerCel is connected to the processor	<b>PowerCel Charge Status</b> <ul style="list-style-type: none"> <li>• 4 quick blinks indicate that the PowerCel is fully charged</li> <li>• 2-3 quick blinks indicate that the battery is sufficiently charged to power the system</li> <li>• 1 quick blink indicates that the PowerCel charge is nearly depleted</li> <li>• If the Orange LED doesn't light up at all, the PowerCel is completely depleted</li> </ul>
	Solid	<b>Low Battery Indicator (programmable feature)</b> When the PowerCel is near depletion, a steady ORANGE light will illuminate indicating that it is time to change the PowerCel.
	Blinks twice every three seconds	<b>Change Battery Indicator (programmable feature)</b> When the PowerCel is depleted to the point that it is unable to support stimulation but not completely depleted, the ORANGE light will blink twice every 3 seconds. Once the battery is fully depleted, no light will emit from the processor and no sound will be transmitted to your implant.
Red	Blinks approximately once every second	<b>Lock Status (programmable feature)</b> When communication between the processor and implant cannot be achieved or has been lost, the RED light will blink approximately once every second. Once your headpiece is properly positioned on your head, and information is transmitting to and from your implant, the flashing will stop.
	Rapidly blinking	<b>Intellilink (applies to CII/HR90K recipients only)</b> Processor is connected to the wrong implant.
	Solid	<b>Processor Error Indicator</b> This indicates a potential problem with your Harmony Processor. If this should occur, first remove and replace your power source. If this does not resolve the problem, please contact your audiologist for additional troubleshooting.
Green	Flickers in response to loud inputs	<b>Microphone/System Status (programmable feature)</b> When the battery and lock sequences are complete, the light should flash GREEN with loud inputs (e.g. when you snap your fingers or speak near to the headpiece microphone) to indicate normal operation.

## Labeling

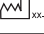The symbols below are used on the labeling for the product and for transportation, and their meanings are as follows:

<b>CAUTION:</b> Federal law restricts this device to sale, distribution and use by or on the order of a physician			
 European Community Mark of Conformity Authorized to affix the CE Mark in 2006			
<b>REF</b>	<b>Model Number</b>	<b>SN</b>	<b>Serial Number</b>
	See Instructions for Use		Store at temperatures between -20° C and +55° C
	Fragile		Date of Manufacture
	Type of Protection: BF		Do not get wet
	Dispose of properly		

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9050050217 Rev A

## Notes

Area for handwritten notes with horizontal dashed lines.

# Notes

A series of horizontal dotted lines for writing notes, filling the page below the title.

[www.AdvancedBionics.com](http://www.AdvancedBionics.com)

#### **NORTH AMERICA**

Advanced Bionics, LLC  
12740 San Fernando Road  
Sylmar, CA 91342 USA  
877.829.0026 in US and Canada  
800.678.3575 TTY  
661.362.1400  
661.362.1500 Fax  
info@AdvancedBionics.com

#### **EUROPE**

Advanced Bionics SARL  
76 rue de Battenheim  
68170 Rixheim, France  
+33.3.89.65.98.00  
+33.3.89.65.50.05 Fax  
europe@AdvancedBionics.com

#### **ASIA-PACIFIC**

Advanced Bionics  
Asia-Pacific Limited  
Suite 4203, 42/F, Tower One  
Lippo Centre, 89 Queensway  
Hong Kong  
852.2526.7668  
852.2526.7628 Fax  
AP@AdvancedBionics.com

#### **LATIN AMERICA**

Advanced Bionics  
28515 Westinghouse Place  
Valencia, CA 91355 USA  
661.362.1723  
661.362.4604 Fax  
LA@AdvancedBionics.com