Advanced Bionics

MRI Safety Information

English, Français, Español, Deutsch, Nederlands, Português

Labeling symbols and their meanings:

 Model number	 Date of manufacture
 Lot Number	 Manufacturer
 Do Not Reuse	 See Instructions for Use
	MR Conditional: Specific circumstances are required for the patient to undergo an MRI procedure with this device in place. Read MRI Safety Information document completely prior to undergoing an MRI.

MRI Safety Information

Testing has demonstrated that the HiRes 90K™ cochlear implant family is MR Conditional. Unilateral and bilateral recipients with this device can be safely scanned in an MR system meeting the following conditions:

- Horizontal closed bore scanners with a static magnetic field of 1.5T (magnet in place or removed)
- Maximum MR system reported, whole body averaged specific absorption rate (SAR) of ≤ 1.7 W/kg at 1.5T for quadrature transmit RF body coils
- Maximum MR system reported, head averaged SAR of ≤ 1 W/kg at 1.5T for quadrature transmit RF head coils
- RMS gradient field of 30 T/s and peak gradient field of 150 T/s
- Maximum spatial field gradient of 2.5 T/m at 1.5T with the magnet in place and 2.5 T/m at 1.5T with the magnet removed.

When tested under the scan conditions defined above with the patient's head in the center of the MR scanner, the HiRes 90K Advantage implant produced a maximum temperature rise of $<2.4^{\circ}\text{C}$ after 15 minutes of continuous scanning when using a body coil and whole body averaged SAR of 1.7 W/kg.

In MRI testing of unilateral recipient conditions, the image artifact caused by the device extends from the HiRes 90K Advantage implant:

- Approximately 6 cm in a 1.5T MRI with the temporary non-magnetic plug in place using a gradient echo pulse sequence
- Approximately 11.5 cm in a 1.5T MRI with the magnet in place using a gradient echo pulse sequence

These artifacts may result in a loss of diagnostic information in the implant vicinity.

MRI Warnings:

Do not allow patients with a HiRes 90K/Advantage cochlear implant to be in the area of an MRI scanner unless the following conditions have been met:

- ***The bandaging protocol recommended by Advanced Bionics is followed when the patient undergoes a 1.5T MRI procedure with the magnet left in place, or***
- ***For cases that would clinically benefit from reduced device artifact (for example some head or neck scans), the internal magnet is surgically removed and possibly replaced with the temporary non-magnetic plug before the patient undergoes an MRI procedure.***

- *The external sound processor and headpiece are removed before entering a room where an MRI scanner is located.*
- *The recommended minimum duration of time post implant surgery prior to undergoing an MRI scan is 2 to 4 weeks in order to allow any inflammation to subside.*
- *An MRI scan is not recommended if the patient has a fever.*
- *Caution: The bandaging protocol with use of the MRI Antenna Coil Cover was developed and approved to prevent magnet displacement and counteract magnet torque during a 1.5T MRI procedure, but some discomfort and pain at the implant site may still be experienced. Please consult with your physician if this is an issue*
- *Caution: If discomfort persists following an MRI, please notify your physician*
- *Caution: Failure to secure the MRI Antenna Coil Cover and internal magnet in place during MRI may result in magnet displacement or the need for surgical revision*
- *Caution: Please consult with your physician prior to MRI to determine if the benefits of MRI are worthwhile over other imaging techniques*
- *MRI preparations must be conducted outside of the MRI room.*
- *Verify that the implant, or both implants if bilaterally implanted, are compatible for conducting an MRI before proceeding. Failure to do so can lead to device movement, device damage, magnet movement, patient discomfort, or trauma and pain to the patient.*
- *MRI procedures are contraindicated for CLARION (C1 and CII) Cochlear Implant recipients.*

For additional information regarding the use of an MRI scanner with a HiRes 90K/Advantage device, please contact Advanced Bionics Technical Support.

MRI testing of the HiRes 90K cochlear implant family with the internal magnet in place is only available in markets where regulatory approval has been received. Contact your Advanced Bionics representative for more information.

MRI Antenna Coil Cover

The MRI Antenna Coil Cover is intended to be used along with a bandaging protocol to allow the patient to undergo an MRI procedure with the magnet left in place. Instructions for use must be followed, including the recommended bandaging protocol.

Compression Dressing prior to MRI Procedure with HiRes 90K Users

This document will provide general guidance for applying appropriate compression dressing over the site of the implanted device (and magnet) in order to perform an MRI procedure.

Required Materials

- Hurt-Free Wrap – Hospital Grade
Latex Free
5 cm width
Johnson and Johnson (or equivalent)
- Flexible tape
Latex Free
2 – 2.5 cm width

3M – Nexcare (or equivalent)

- Patient Headpiece
- MRI Antenna Coil Cover (CI-7521)
Advanced Bionics
- Marking Pen


Procedure

1. Place patient in sitting position to allow access to the implant site.

NOTE: If the patient is bilaterally implanted, prepare to locate both implant devices using the following steps. Both implant locations must have compression dressing placed over their magnets.

Secure MRI Antenna Coil Cover over implant magnet site

2. Place the patient headpiece (with cable removed) over the implant site. The magnets will hold the headpiece in place.

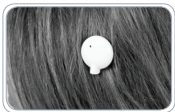
Patient headpiece magnetically attached

3. Cut a piece of the Coach bandage that is long enough to wrap around the head once.
4. Wrap this piece around the head, so that the bandage covers the patient headpiece. Make this wrap tight.

Patient headpiece under bandage

5. Outline the position of the headpiece on the bandage using a marker or pen.

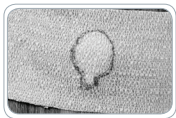Marked bandage showing outline of headpiece

6. Slip out the patient headpiece, but keep the bandage in place.

Removing headpiece with bandage in place

7. Slide the MRI Antenna Coil Cover under the bandage, lining it up with the marked outline of the headpiece.

Inserting MRI Antenna Coil Cover

MRI Antenna Coil Cover under marked outline

Measure head size and bandage length needed for compression wrapping

8. Take the remaining Coach bandage roll, and wrap the bandage around the head once, without stretching.

Remaining Coach bandage roll wrapped once around head

9. Mark the location on the bandage that is one full wrap around the head. This is the head circumference.

Mark bandage one full wrap around head

Marked head circumference


10. Unwrap the piece of bandage with the marked head circumference, and place it on a flat surface.


Marked head circumference

11. Unroll the remaining bandage roll.

Fold over the bandage start and crease at the marked head circumference.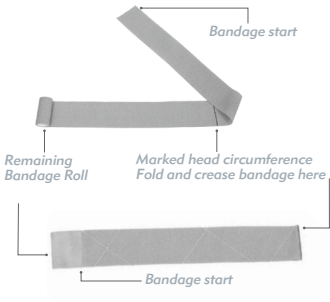

12. Cut or tear the remaining bandage where it overlaps with the bandage start. The resulting bandage piece length is twice the head circumference.

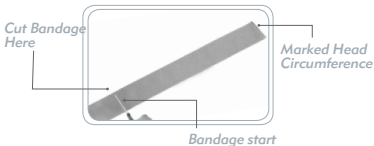Apply compression bandaging

13. Wrap this cut piece, this time very tightly, by stretching the marked line an additional half turn around the head. This ensures 150% extension of the bandage.

NOTE: After 1.5 turns, the marked line should be on the opposite side of the head from where you started the wrap.

14. Continue wrapping the bandage at 150% extension for an additional 1.5 turns, resulting in 3 full turns total.

NOTE: After 3 turns, the bandage should end where you started the wrap (further is acceptable also).

After the MRI

Once the MRI procedure has been completed, gently remove the Coach Wrap, tape, and the MRI Antenna Coil Cover.

Discard all components including the MRI Antenna Coil Cover.

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© 2019 Advanced Bionics AG and affiliates. All Rights Reserved.
Advanced Bionics AG
Advanced Bionics LLC
California, U.S.A.