ADVANCED
BIONICS

POWERFUL CONNECTIONS

A Sonova brand

Zn-Air Battery Pak

CI-5500

instructions for use

English, Français, Español, Deutsch, Nederlands, Italiano

CONTENTS

SYMBOLS TABLE	3
PURPOSE AND SCOPE OF IFU	5
LIMITATIONS AND CONTRAINDICATIONS	5
PRECAUTIONS, CAUTIONS, AND WARNINGS	6
UNDESIRABLE SIDE EFFECTS	7
PRODUCT DESCRIPTION AND PERFORMANCE CHARACTERISTICS	8
INDICATIONS FOR USE	8
INTENDED PURPOSE	10
<i>Intended Users</i>	<i>11</i>
<i>Intended Use Environment</i>	<i>11</i>
COMPATIBILITY	11
OPERATING INSTRUCTIONS	12

CARE AND MAINTENANCE	16
<i>Expected Life</i>	17
DISPOSAL AND HAZARDOUS MATERIALS	17
AVAILABLE PRODUCT OPTIONS AND MODEL NUMBERS	18
CLINICAL BENEFITS	20
SUMMARY OF SAFETY AND CLINICAL PERFORMANCE	20

SYMBOLS TABLE

	European Community Mark of Conformity
	Model number
	Date of manufacture
	Manufacturer
	Type of Protection: BF
	Suitable temperature range for transport and storage
	Fragile
	Do not get wet
	Lot Number

	See Instructions for Use
	Dispose of in accordance with applicable national and local regulations
EC REP	European authorized representative
UDI	Unique Device Identifier
MD	Medical Device
	Caution (refer to "Precautions, Cautions and Warnings" section for further information)
	MR Unsafe
IPxx	Ingress Protection Rating

PURPOSE AND SCOPE OF IFU

This IFU is designed to help cochlear implant recipients, and their caregivers if applicable, to understand the use and care of the Zn-Air Battery Pak. It is also meant to be used by hearing care professionals trained in the fitting of cochlear implants to counsel the recipients on the use of the Zn-Air Battery Pak and to troubleshoot problems.

LIMITATIONS AND CONTRAINDICATIONS

As the Zn-Air Battery Pak is a component of the HiResolution™ Bionic Ear System, the following contraindications stated for the HiResolution Bionic Ear System are applicable: deafness due to lesions of the acoustic nerve or central auditory pathway; cochlear ossification that prevents electrode insertion; absence of cochlear development; active external or middle ear infections; tympanic membrane perforations associated with recurrent middle ear infections.

PRECAUTIONS, CAUTIONS, AND WARNINGS

- *This device should be used only by the individual for whom it is prescribed.*
- *CHOKING HAZARD: The Zn Air Battery Pak contains small parts that pose a hazard of inhalation choking, or ingestion. If any parts are swallowed or inhaled, consult a physician or hospital immediately.*
- *Do not leave children unattended with or allow children to play with the Zn-Air Battery Pak or battery cells.*
- *Do not allow children to place the Zn-Air Battery Pak or battery cells in the mouth or nose*
- *Do not connect the Zn-Air cartridge to the charger. The charger is to be used with the rechargeable Lithium Ion PowerCels only.*
- *Remove your sound processor and consult your cochlear implant professional if uncomfortable sounds are heard or in case of discomfort, pain or skin irritation.*
- *If a battery shows visible signs of leakage, dispose of it in accordance with local regulations.*

- *Do not allow leaking battery fluid to come into contact with skin, mouth, or eyes, as this fluid may cause a burn.*
- *High levels of static electricity may damage electronic components of your sound processor or implant. Care should be taken to avoid exposure of the system to static electricity.*
- *In the event that you experience any issues with your product, please contact your cochlear implant professional or the manufacturer. Do not attempt to service or modify the Naída CI or its accessories. Doing so may compromise system performance and will void the manufacturer's warranty. Products should be serviced only at Advanced Bionics and damaged products should be returned to Advanced Bionics.*

UNDESIRABLE SIDE EFFECTS

Undesirable side effects of the Naída™ CI system may include skin irritation and discomfort from pressure on the ear, device overheating, or overly loud sounds.

PRODUCT DESCRIPTION AND PERFORMANCE CHARACTERISTICS

The Zn-Air Battery Pak provides a disposable battery source for the Naída CI sound processor. The type of disposable battery to be used with Naída CI is size 675 Power Implant batteries (675P). Regular size 675 hearing instrument batteries will not be powerful enough for a cochlear implant. The suggested manufacturer and battery is PowerOne P675 Implant Plus battery.

INDICATIONS FOR USE

The Zn-Air Battery Pak is an external component of the HiResolution Bionic Ear System, which is intended to restore a level of auditory sensation to individuals with severe-to-profound sensorineural hearing loss via electrical stimulation of the auditory nerve.

Adults

- *18 years of age or older.*
- *Severe-to-profound bilateral sensorineural hearing loss or severe-to-profound unilateral hearing loss.*
- *Post-lingual onset of severe or profound hearing loss.*

- *Limited benefit from appropriately fitted hearing aids, defined as scoring 50% or less on a test of open-set sentence recognition (HINT Sentences).*

Children

- *12 months through 17 years of age.*
- *Severe-to-profound bilateral sensorineural deafness or severe-to-profound unilateral hearing loss.*
- *Use of appropriately fitted hearing aids for at least 6 months in children 2 through 17 years of age, or at least 3 months in children 12 through 23 months of age. The minimum duration of hearing aid use is waived if x-rays indicate ossification of the cochlea.*
- *Little or no benefit from appropriately fitted hearing aids. In younger children (< 4 years of age), lack of benefit is defined as a failure to reach developmentally appropriate auditory milestones (such as spontaneous response to name in quiet or to environmental sounds) measured using the Infant-Toddler Meaningful Auditory Integration Scale or Meaningful Auditory Integration Scale or $\leq 20\%$ correct on a simple open-set word recognition test (Multisyllabic Lexical Neighborhood Test) administered using monitored live voice (70 dB SPL). In older children (≥ 4 years of age), lack of hearing aid benefit is defined as scoring $\leq 12\%$*

on a difficult open-set word recognition test (Phonetically Balanced-Kindergarten Test) or $\leq 30\%$ on an open-set sentence test (Hearing In Noise Test for Children) administered using recorded materials in the sound field (70 dB SPL).

INTENDED PURPOSE

The Zn-Air Battery Pak is an accessory of an auditory active-implantable system, the HiResolution Bionic Ear system. The HiResolution Bionic Ear system is intended to provide auditory sensation via electrical stimulation of the auditory nerve for individuals with severe to profound bilateral or unilateral sensorineural hearing loss. Severe hearing loss is defined as audiometric thresholds greater than or equal to 70 dB HL, but less than 90 dB HL. Profound hearing loss is defined as audiometric thresholds greater than or equal to 90 dB HL. The external components work together with the implant to bypass the damaged part of the inner ear and convert sound picked up by the microphone or streamed via wireless communication into electrical signals that are used by the cochlear implant to enable hearing.

The Zn-Air Battery Pak is intended to power the Advanced Bionics Naída CI Sound Processors.

Intended Users

The intended users of the Zn-Air Battery Pak are users of the Naída CI Sound Processors, their caregivers if applicable, and hearing care professionals. The recipient, or their caregiver, should, at a minimum, be capable of changing the battery, connecting the earhook, placing and removing the sound processor from their ear, and placing and removing the headpiece from the implant site. The hearing care professional should be trained in the use and fitting of the cochlear implant system.

Intended Use Environment

The intended use environments of the Zn-Air Battery Pak are healthcare and daily living environments. The Zn-Air Battery Pak is designed to be worn daily during the user's normal waking hours.


COMPATIBILITY

The Zn-Air Battery Pak is compatible with Naída CI Sound Processors from Advanced Bionics.

OPERATING INSTRUCTIONS

To attach the battery cartridge to the Naída CI hold the sound processor in one hand. Rotate RF cable upward to avoid bumping it upon battery placement.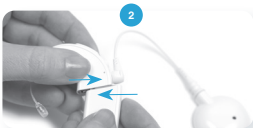

Align the battery cartridge connector with the processor housing side of the connector.

Slide the battery onto the processor until you feel it click into place.

Inserting Zn-Air Disposable Battery Cells (Tamperproof Cartridge)

Using the same tool provided for T-Mic 2 or earhook removal and replacement, insert the pointed end of the tool into the small hole located on the spine of the Zn-Air cartridge.

With the tool inserted in the hole, gently push in and downward until you feel the Zn-Air drawer release, and pop out the button of the Zn-Air cartridge.

Note: If you do not have the tool present, any thin pointed device should work to release the drawer.

You no longer need the tool. From here you can grasp the emerged Zn-Air drawer with your hand and gently pull it the rest of the way out so you are able to insert the Zn-Air cells.

Follow steps 3 to 8 of the previous section 'Inserting Zn-Air Disposable Battery Cells (Non-Tamperproof Cartridge)'.

CARE AND MAINTENANCE

Storing the Zn-Air Battery Pak

When not in use, store Zn-Air Battery Pak in the equipment case provided.

Cleaning and Maintenance

Wipe the outside of the Zn-Air Battery Pak with a soft, dry cloth. To prevent intermittent operation of the sound processor, the battery contacts on the PowerCel Battery Charger, Zn-Air Battery Pak, and on the sound processor should be kept free from dirt and dust. Gently clean the area with a hearing aid brush or dry cotton swab at least once a month. If the Zn-Air Battery Pak is exposed to moisture, place it in a drying system before charging and before next use. If the Zn-Air Battery Pak is dropped, inspect it for evidence of damage or cracking. If any evidence of damage is seen, the Zn-Air Battery Pak should be replaced.

Do not attempt to service or modify the Naída CI or its accessories. Doing so may compromise system performance and will void the manufacturer's warranty. Products should be serviced only at Advanced Bionics.

Expected Life

The expected life of the Zn-Air Battery Pak is 1 year.

Ingress Protection (IP) Rating

The following product configurations have an IP rating of 54 (protection against dust; protection against failure due to water splashes from all directions): (indented bullet)The Naída Q90 Sound Processor with a PowerCel battery or Zn-Air Battery Pak, a T-Mic or Acoustic Earhook or Earhook, RF cable and an IP54 rated headpiece.

DISPOSAL AND HAZARDOUS MATERIALS

For proper disposal of rechargeable batteries in Europe, please return depleted Zn-Air Battery Pak to the nearest AB location or the designated programming/follow-up center.

Recommended Operating and Storage Temperature Ranges

CONDITION	MINIMUM	MAXIMUM
Operating Temperature	0°C (32°F)	45°C (115°F)
Storage Temperature	-20°C (-4°F)	55°C (131°F)

AVAILABLE PRODUCT OPTIONS AND MODEL NUMBERS

DESCRIPTION	MODEL NUMBER
Naída CI Q90 Sound Processor* †	CI-5280-120, CI-5280-130, CI-5280-140, CI-5280-150, CI-5280-160, CI-5280-170, CI-5280-190, CI-5280-110, CI-5280-220, CI-5280-230, CI-5280-240
Naída CI Q70 Sound Processor* †	CI-5245-110, CI-5245-120, CI-5245-130, CI-5245-150, CI-5245-160, CI-5245-170, CI-5245-140, CI-5245-190, CI-5245-220
Naída CI Q30 Sound Processor* †	CI-5260-120, CI-5260-140, CI-5260-150
Universal Headpiece (UHP)*	CI-5305
Concave UHP* †	CI-5307
RF Cable*	CI-5415-301, CI-5415-302, CI-5415-303, CI-5415-304, CI-5415-305, CI-5415-201, CI-5415-202, CI-5415-203, CI-5415-204, CI-5415-205, CI-5415-401, CI-5415-402, CI-5415-403, CI-5415-404, CI-5415-405, CI-5415-501, CI-5415-502, CI-5415-503, CI-5415-504, CI-5415-505

Zn-Air Battery Pak*	CI-5500-110, CI-5500-120, CI-5500-130, CI-5500-140, CI-5500-150, CI-5500-160, CI-5500-170, CI-5500-190, CI-5500-220, CI-5500-240, CI-5500-230
Zn-Air Battery 6pk	070-0328
Zn-Air Battery 6pk, mercury free	070-0329

****Applied part per IEC 60601-1***

†Only available in markets where regulatory approval has been received and the product has been made commercially available. Contact Advanced Bionics for more information.

NOTE: model number extensions denote variants related to product color, size, and/or length.

CLINICAL BENEFITS

The Intended Clinical Benefit of the Zn-Air Battery Pak as part of the HiResolution Bionic Ear System is to:

- *provide useful hearing to individuals with severe-to profound hearing loss via electrical stimulation of the auditory nerve.*
- *provide a combined type of stimulation, electric plus acoustic, when sound processors are enabled with the Acoustic Earhook.*

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

A summary of safety and clinical performance (SSCP) is available in the European database on medical devices (Eudamed), where it is linked to the Basic UDI-DI:- 08400944CI5500XU
<https://ec.europa.eu/tools/eudamed>

Users of the Zn-Air Battery Pak in the European Union should report any serious incident to their local competent authority and to Advanced Bionics.

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©2023 Advanced Bionics LLC. All Rights Reserved.
Advanced Bionics LLC
Advanced Bionics GmbH
AB – A Sonova brand